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Dual Rinse® HEDP Mild chelator for endodontic irrigating solutions (En)

INSTRUCTIONS FOR USE

- USE: **Dual Rinse® HEDP** is an additive for endodontic irrigating solutions.
 - Dual Rinse® HEDP** should be added to chemically neutral or sodium hypochlorite rinsing solutions.
 - The dentist will decide on the indication and use of **Dual Rinse® HEDP**.
- CONTRAINDICATIONS: Do not use **Dual Rinse® HEDP** in patients with reported sensitivity to the product.
- WARNINGS: None known.
- PRECAUTIONARY MEASURES:
 - Dual Rinse® HEDP** packages (capsules) must be tightly sealed and stored in a dry place to prevent loss of efficacy due to contamination by moisture.
 - A rubber dam should be used during the treatment.
 - Do not use at the same time with chlorhexidine and EDTA.
 - Do not use at the same time with >5% sodium hypochlorite.
 - Do not heat together with sodium hypochlorite.
 - Do not flush the endodontic irrigating solution through the apex.
- ADVERSE EFFECTS: None known.
- USE OF **Dual Rinse® HEDP**
 - A cup (mixing cup) should be sterilized in the autoclave in accordance with dental specifications ("Preparation of sterile medical products"). Place 10 ml of sodium hypochlorite (NaOCl) into the sterilized cup.
 - Open a capsule and place the powder from one capsule of **Dual Rinse® HEDP** into the sterilized mixing cup with NaOCl.
 - Stir the **Dual Rinse® HEDP** powder with a sterile instrument until it has completely dissolved.
 - Use the NaOCl / **Dual Rinse® HEDP** solution in accordance with the endodontic rinsing protocol.
 - The prepared solution is intended for single and immediate use only.

Material Safety Data:

- HANDLING: This product is intended for dental use only. If the capsules have been opened, discard the material. Do not let any moisture get into the capsule container.
- STORAGE: Store in a dry place.
- TOXICITY: The powder is caustic!
Do not swallow, inhale or bring in contact with your eyes or other mucous membranes.
If it gets into your eyes or mucous membranes, rinse with plenty of water.
- DISPOSAL: **Dual Rinse® HEDP** can be safely disposed of with household waste. Do not put it into the sewage system.

Warning: It is important to comply with the instructions for use!

Store in a cool and dry place.

Discard opened capsules.

Do not use after the expiration date! (= EXP)

In the event of questions or incidents, please send an e-mail to info@medcem.ch

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Medcem GmbH, Bahnhofstrasse 12, 8570 Weinfelden, Suiza www.medcem.ch