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
**INSTRUCTIONS FOR USE**

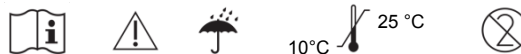1. USE: **MED-PZ** use only for dental endodontics. Use in personal responsible indication.
2. CONTRAINDICATIONS: Do not use **MED-PZ** in patients with reported sensitivity to the product.
3. WARNINGS: Only use **MED-PZ** when the acute phase of endodontic inflammation has been overcome. Exudate from periapical tissue can interfere with hardening. Do not mix with NaCl (0.9% solution).
4. NOTE: Does not differ from dentin on X-ray. **MED-PZ** should be mixed with distilled or sterile water.
5. PRECAUTION:
  - a. **MED-PZ** packs (capsules) must be tightly closed and stored in a dry place to avoid a loss of effectiveness due to moisture.
  - b. **MED-PZ** must be used immediately after mixing to avoid drying out while it sets.
  - c. Use rubber dam, gloves and eye protection during use.
6. ADVERSE REACTIONS: May cause discolouration of the treated teeth due to bleeding.
7. **MED-PZ** MIXING INSTRUCTIONS
  - a. Open a capsule and dispense the entire content onto a sterile mixing pad.
  - b. Mix small amounts of water into the powder to obtain a paste-like and clayey consistency. (~ 0.12 ml water per capsule).
  - c. Mix the material with the water for max. one minute to ensure all the powder particles are hydrated.
  - d. Adding too much or too little water will reduce the ultimate strength of the cement.
  - e. The working time is a maximum of 5 minutes and Medcem MTA then hardens. **MED-PZ** sets over a period of 4 hours initially. The final hardness is only obtained after about 28 days.

**Material Safety Data**

1. HANDLING: This product is intended for dental use only. If the capsules have been opened, discard the material. Do not let any moisture get into the capsule container.
2. STORAGE: Store in dry conditions.
3. TOXICOLOGICAL INFORMATION: Powder may cause strong irritation. Do not incorporate by swallowing or breathing in the material. Do not bring in contact to eyes or mucous membrans.  
In case of contact use water to rinse.
4. DISPOSAL CONSIDERATIONS: This product can be disposed of with household waste. Do not allow to enter sewers water.

**Warning: It is important to comply with the instructions for use!**  
**Store in a cool and dry place.**  
**Discard opened capsules.**  
**Do not use after the expiration date! (= EXP)**  
**In the event of questions or incidents, please send an e-mail to info@medcem.ch**

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**Medcem GmbH**, Bahnhofstrasse 12, 8570 Weinfelden,